

Master Plus Discolored Black phenomenon

After Staining or Glazing seems like this black surfaces of teeth?



Please check follows

1. Clean inside chamber? no burned dust inside? Also, clean the firing tray and pegs?
2. please check the appropriate programming? follow the original schedule?
Correct temperature and time? also, correct materials using?
3. The same company's liquid using for staining material when the mixing?
4. One material company says that please don't mix use the other company's liquid, that leads to turn the discolored to black.
5. There is no AC air duct, Gas Burner or Wax burnout machine surround the Master Plus?
6. Summary: this black colored phenomenon is not from Master Plus machine itself.
 - a. Please check again the correct technique and material using.