Semi/Non-Precious (PALLADIUM MORE THAN 30%)

1 Turn on the power swich on your right side	
 Choose your course by 1. <u>START TEMP</u> will flash 2. <u>START TEMP</u> illuminates 	(0 to 99)
3 Put the alloy into <u>CERAMIC CRUCIBLE</u>	
4 Put <u>CERAMIC CRUCIBLE</u> into <u>CRUCIBLE</u>	COUNTER
Press CAST CYCLES	
5 Put CERAMIC CRUCIBLE into the muffle	
Press MELT Double click Argon FLOW 1. Argon flow nozzile comes down 2. CAST TEMP will flash 3. HOLD TIME will illuminates	*DON'T PRESS button (AIR has to be on) *ARGON GAUGE (a red ball) has to be 1.5
6 Machine will beep	
Press CAST Argon flow nozzile goes back in	
Put your burned-out casting ring on the muffle	
8 Close the chamber lid and move the locking arm down to the bottom	
Chamber will invert then revert automatically	
9 Done	

Open the chamber lid